

# Dressing Application Guide

## Invia® Foam Dressing Kit with FitPad



**1. Thoroughly clean and debride the wound.**  
Apply skin prep to protect the peri-wound skin.



**2. If required, apply a non-adherent contact layer** prior to placing the foam into the wound.



**3. Cut the foam** to fit the size and shape of the wound. Do not cut directly over the wound.



**4. Rub the edges** of the foam to remove any loose particles.




**5. Place the foam** into the wound cavity. Do not tightly pack or force foam into areas of the wound. Do not allow the foam to overlap onto intact skin.



**6. Apply transparent film.** Partially peel back one side of layer 1 and place the adhesive side down. Remove the remaining side of layer 1.



**7. Now remove** the transparent backing layer labeled 2.

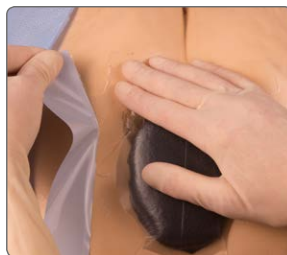


**8. Remove the** perforated silver colored tab on the side. The film should extend 3–5 cm beyond the wound margin to facilitate adequate seal.



**9. Select** appropriate location for the suction pad (FitPad) to be applied. **Pinch the film** and cut a small hole (approx. 1 cm).




**10. Peel the** backing off the FitPad to expose the adhesive.

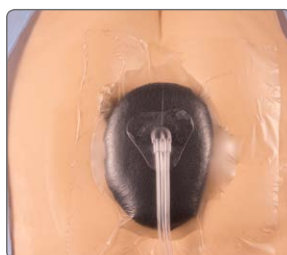



**11. Place the FitPad** centered over the previously cut hole in the dressing. Press firmly for adherence.



**12. Attach the** dressing tubing to the pump tubing by pushing the Quick-connector together until you hear a click.


 The Invia Foam Dressing Kit with FitPad is intended to be used in conjunction with the Invia Motion and Invia Liberty Negative Pressure Wound Therapy Systems (NPWT).

 Invia NPWT instructions advise 24 hour therapy without interruption. If therapy is discontinued for more than 2 hours, the dressing should be replaced and therapy restarted by a healthcare professional.


 This Quick Card does not replace the instructions for use REF 200.8923.

# Dressing Application Guide

Invia® Gauze Dressing Kit with FitPad


- 


1. Thoroughly clean and debride the wound. Apply skin prep to protect the peri-wound skin.
- 

2. If required, apply a non-adherent contact layer prior to placing the gauze into the wound.
- 


3. Cut the gauze to fit the size and shape of the wound. Do not cut the gauze directly over the wound.
- 

4. Saturate the gauze with the saline provided.
- 

5. The gauze should fit loosely into the wound bed. Do not tightly pack or force gauze into areas of the wound.
- 


6. Apply transparent film. Partially peel back one side of layer 1 – place the adhesive side down. Remove the remaining side of layer 1.
- 

7. Now remove the transparent backing layer labeled 2.
- 

8. Remove the perforated silver colored tab on the side. The film should extend 3–5 cm beyond the wound margin to facilitate an adequate seal.
- 

9. Select appropriate location for the suction pad (FitPad) to be applied. Pinch the film and cut a small hole (approx. 1 cm).
- 

10. Peel the backing off the FitPad to expose the adhesive.
- 

11. Place the FitPad centered over the previously cut hole in the dressing. Press firmly for adherence.
- 

12. Attach the dressing tubing to the pump tubing by pushing the Quick-connector together until you hear a click.

The Invia Gauze Dressing Kit with FitPad is intended to be used in conjunction with the Invia Motion and Invia Liberty Negative Pressure Wound Therapy Systems (NPWT).

Invia NPWT instructions advise 24 hour therapy without interruption. If therapy is discontinued for more than 2 hours, the dressing should be replaced and therapy restarted by a healthcare professional.

This Quick Card does not replace the instructions for use REF 200.9065.

Medical Vacuum Technology  
for Healthcare Professionals

Please contact us or your local Medela  
representative for details.

Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
www.medela.com

**International Sales**  
Medela AG  
Lättichstrasse 4b  
6341 Baar  
Switzerland  
Phone +41 41 562 51 51  
Fax +41 41 562 51 00  
www.medela.com

**United Kingdom**  
Medela UK Ltd.  
Huntsman Drive  
Northbank Industrial Park  
Irlam, Manchester M44 5EG  
United Kingdom  
Phone +44 161 776 0400  
Fax +44 161 776 0444  
info@medela.co.uk  
www.medela.co.uk

Local contact: