Invia® – Silverlon NPWT Apositos Antimicrobianos Contacto para Heridas

Español

Invia® – Silverlon NPWT

EN Antimicrobial Wound Contact Dressing - Instructions for use

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Device Description

Invia-Silverlon® NPWT Wound Contact Dressings are sterile, single layer, nonadherent, antimicrobial barrier dressings, intended for topical for use in negative pressure wound therapy (NPWT), to reduce risk of wound infection.

Available as:

Invia-Silverlon® NPWT Wound contact Dressing

REF #	Description	Size
087.7011	Invia® -Silverlon NPWT Negative Pressure Dressing 10 x 12 cm	10 x 12 cm (WCDNPS-45)
087.7012	Invia® -Silverlon NPWT Negative Pressure Dressing 12 x 20 cm	12 x 20 cm (WCDNPS-58)

Invia-Silverlon® NPWT Wound Contact Dressings contain silver to provide an ancillary, local antimicrobial effect to reduce the risk of wound infection by organisms sensitive to silver. Ionic silver is a broad spectrum antimicrobial, which has been shown to be effective against a range of pathogens associated with wound infection. Invia-Silverlon® NPWT dressings have been tested in vitro and found effective against microorganisms such as: Enterococcus faecalis, Staphylococcus aureus, Pseudomonas aeruginosa, Acinetobacter Baumanni, Enterobacter cloacae, Staphylococcus epidermidis und Klebsiella pneumoniae.

Invia-Silverlon® NPWT Wound Contact Dressings may be used with all Invia negative pressure wound therapy porous foam/gauze dressings and equipment.

In vitro testing has shown Invia-Silverlon® NPWT dressings to be MRI compatible.

Indications

Invia-Silverlon® NPWT Wound Contact Dressings are professional use wound dressings indicated for use in conjunction with Invia negative pressure wound therapy (NPWT), for local management of:

- 1st and 2nd degree burns
- Donor and graft site wounds
- Traumatic and surgical wounds
- Diabetic, pressure and venous ulcers
- Infected wounds*

*Invia-Silverlon® NPWT Wound Contact Dressings may be used on those infected wound types listed above. Where used on infected wounds, the infection should be treated as per local clinical protocol.

Invia-Silverlon® NPWT Wound Contact Dressings are indicated for a total duration of contact (i.e. where treatment involves consecutive application of individual dressings) of up to 30 days.

Warnings

- Do not use past expiration date on the product packaging.
- Do not use if pouch is damaged or open.
- Clinicians / Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver-containing dressings, particularly in children and neonates.
- Invia-Silverlon® NPWT Wound Contact Dressings should not be used during pregnancy or lactation because of the lack of data concerning the effects of silver on reproduction.
- Invia-Silverlon® NPWT Wound Contact Dressings are intended for single use only and should not be re-used. Re-use may adversely affect the dressing's performance characteristics and also presents a risk of infection to the patient.
- Do not use petroleum-based ointments or creams under Invia-Silverlon® Wound Contact Dressings.
- Do not moisten Invia-Silverlon® NPWT Wound Contact Dressings with hydrogen peroxide or povidone iodine.
- Total duration of use of Invia-Silverlon® NPWT Wound Contact Dressings (i.e. where treatment involves consecutive application of individual dressings) should not exceed 30 days.
- Invia-Silverlon® NPWT Wound Contact Dressings are not intended for direct application to exposed underlying structures (i.e. blood vessels, bone, tendons, nerves).

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© Medela AG/201.0959/2017-06/B